THUASNE

Podosil

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Full-length soft-cushion insole

Purpose

Podosil is a full-length soft insole made of silicone that gently cushions the entire foot. The inlay may only be used for foot treatment.

Mode of action

Full-length Podosil insoles support the foot and relieve pressure in the sensitive toe and heel areas. The special honey-comb structure increases this effect. The contoured shape helps in the management of heel spurs. Podosil provides light longitudinal and lateral arch support. The visco-elastic material, with its variable compression rate, promotes a massaging effect which in turn stimulates blood circulation. The skin-friendly cotton finish ensures easy access and a comfortable environment for the foot.

Indications

Collapsed arches, Over-usage syndrome, Arthrosis, Heel spur, Relief of ankle, knee and hip implants.

Contraindications

Ulcers in the forefoot and the heel area.

Side Effects

None reported when applied and worn as intended.

How to measure

- Select the right insoles depending on shoe size.
- Should Podosil insoles prove too big for the shoe, cut to size.
- Insert insoles into the shoe (R = right, L = left).
- The chosen footwear should be of such a size that, even after the insoles have been fitted, sufficient room for the feet remains.
- Please note: It may be necessary to get used to wearing Podosil insoles for 2-3 days, before they feel really comfortable.

Precautions

Follow the advice of the professional who prescribed or supplied the product. Consult your professional if discomfort occurs. Store at room temperature, preferably in its original packaging. For hygiene and performance reasons, do not re-use the product for another patient. Any serious incident that occurs in connection with the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient are/is domiciled.

Care

Podosil insoles can be washed by hand, using a mild detergent.

Material composition

Silicone, polyamide, cotton

Please keep this instruction leaflet.

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- IL** THUASNE CERVITEX Ltd. · The Bell House, 20 Hataas St. · Kfar Saba, 44425
- S** THUASNE SCANDINAVIA AB · Box 95 · 131 07 Nacka
THUASNE DEUTSCHLAND GmbH