THUASNE

Pedipro Plus


fr	Semelles silicone avec inserts anatomiques	5
en	Silicone insoles with anatomical inserts	6
de	Silikoneinlagen mit anatomischen Pelotten	8
nl	Siliconen inlegzolen met anatomische inserts	10
it	Solette in silicone con inserti anatomici	11
es	Plantillas de silicona con insertos anatómicos	13
pt	Palmilhas de silicone com inserções anatómicas	14
da	Såler i silikone med anatomiske indsats	16
fi	Anatomisilla tyynyillä varustetut silikoni-irtopohjalliset	18
sv	Silikonsulor med anatomiska delar	19
el	Πάτσι σιλικόνης με ανατομικά ενθέματα	21
cs	Silikonové vložky s anatomicky tvarovanými vloženými prvky	23
pl	Podeszwy silikonowe z wkładkami anatomicznymi	24
lv	Silikona zoles ar anatomiskiem ieliktniem	26
lt	Silikoniniai vidpadžiai su anatominiais įdėklais	27
et	Silikoost sisetallad anatoomiliste sisepadjanditega	29
sl	Silikonski podplati z anatomskimi vložki	31
sk	Silikonové vložky s anatomickými výstelkami	32
hu	Szilikon talpbetét anatómiai betétekkel	34
bg	Силиконови подметки с анатомични вложки	36
ro	Branțuri din silicon cu inserții anatomice	37
ru	Силиконовые стельки с анатомическими вставками	39
hr	Silikonski ulošci s anatomskim umetcima	41
zh	带有符合解剖形状插片的硅胶鞋垫	43
ar	نعال سيليكونية مزودة بحشوات متوافقة مع شكل القدم	47

		Europe	UK
	1	37 - 39	4 - 5 ^{1/2}
	2	40 - 41	6 ^{1/2} - 7
	3	42 - 43	8 - 8 ^{1/2}
	4	44 - 46	9 ^{1/2} - 11

fr	Pointure - Europe - UK
en	Shoe size - Europe - UK
de	Schuhgröße - Europa - UK
nl	Schoenmaat - Europa - VK
it	Numero di scarpa - Europa - Regno Unito
es	Número de pie - Europa - Reino Unido
pt	Número do sapato - Europa - Reino Unido
da	Størrelse - Europa - UK
fi	Kengän koko - Eurooppa - Yhdistynyt kuningaskunta
sv	Skostorlek - Europa - Storbritannien
el	Νούμερο υποδημάτων - Ευρώπη - ΗΒ
cs	Velikost obuvi - Evropa - Spojené království
pl	Numer buta - Europa - Wielka Brytania
lv	Apavu izmērs - Eiropa - Apvienotā Karaliste
lt	Avalynės dydis - Europa - JK
et	Suurus - Euroopa - Ühendkuningriigid
sl	Velikost obutve - Evropa - UK
sk	Veľkosť - Európa - Spojené kráľovstvo
hu	Cipőméret - Európa - UK (Egyesült Királyság)
bg	Размер - Европа - Обединено кралство
ro	Mărimea pantofului - Europa - Regatul Unit
ru	Размер обуви - Европа - Великобритания
hr	Broj cipela - Europa - UK
zh	鞋码-欧洲-英国
ar	المقاس - أوروبا - المملكة المتحدة

Décharge	Restaure l'équilibre	Correction biomécanique
Off-loading	Balance restoration	Biomechanical correction
Entlastung	Stellt das Gleichgewicht wieder her	Biomechanische Korrektur
Losmaken	Herstelt het evenwicht	Biomechanische correctie
Scarico	Restaure l'equilibrio	Correzione biomeccanica
Descarga	Restaure el equilibrio	Corrección biomecánica
Alívio do peso	Restaure o equilíbrio	Correção biomecânica
Aflastning	Genopretter ligevægten	Biomekanisk korrektion
Vaputus	Palauttaa tasapainon	Biomekaaninen korjaus
Avlastning	Förbättrar balansen	Biomekanisk korrigering
Αποφόρτιση	Αποκαθιστά την ισορροπία	Βιο-μηχανικός έλεγχος
Snížení zátěže	Obnovuje rovnováhu	Biomechanická korekce
Odciążenie	Produkt przywraca równowagę	Korekcja biomechaniczna
Atbrīvošana	Līdzsvara atjaunošana	Biomehāniskā korekcija
Apkrovis sumažinimas	Pusiausvyros palaikymas	Mechaninė biologinė korekcija
Koormuse vähendamise	Taastab tasakaalutunnetuse	Biomehaaniline korrektsioon
Razbremenitev	Vzpostavi nazaj ravnovesje	Popravek biomehanske motnje
Uvoľnenie	Obnovenie rovnováhy	Biomechanická korekcia
Tehermentesítés	Az egyensúly helyreállítása	Biomechanikus korrekció
Отбременяване	Възстановява баланса	Биомеханична корекция
Descărcare	Restabilirea echilibrului	Corecție biomecanică
Разгрузка	Восстановление равновесия	Биомеханическая коррекция
Rasterečenje	Uspostavljanje ravnoteže	Biomehānička korekcija
减压	恢复平衡	生物力学矫正
تخفيف ثقل	إعادة التوازن.	تصحيح ميكانيكي حيوي

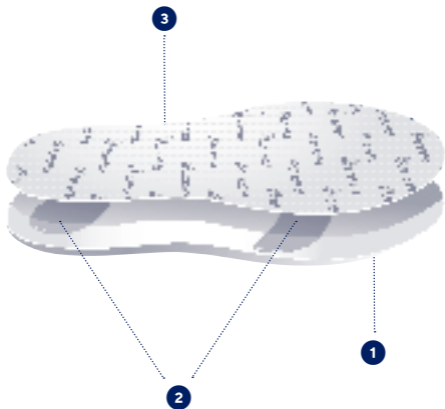fr

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en

SILICONE INSOLES WITH ANATOMICAL INSERTS

Description/Destination

Silicon provides even distribution of pressure and shock absorption. ❶
Blue, flexible, anatomically-shaped inserts in the area of the heel and metatarsals. ❷

Components promoting hygienic freshness. ❸

Composition

Silicone - polyamide - silver.

Properties/Mode of action

Off-loading.

Balance restoration.

Biomechanical correction

Indications

Heel and foot pain (metatarsalgia, plantar fasciitis).

Rheumatoid foot.

Relief for tendinitis.

Contraindications

Do not apply the product in direct contact with broken skin or an open wound without an adequate dressing.

Do not use in the event of known allergy to any of the components.

Precautions

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

Choose the appropriate size to fit, referring to the size chart.

For hygiene and performance reasons, do not re-use the product for another patient.

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Do not bring the product into direct contact with greasy substances (ointment, cream, etc.).

Undesirable side-effects

This device can cause skin reactions (redness, itching, irritations...).

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

Place the insole flat in the shoe, pressing the rim of the product against the heel of the shoe.

N.B.: this product is designed for a specific side, indicated by "RIGHT" for your right shoe and "LEFT" for your left shoe.

Repeat this operation with the other insole.

The foot must rest on the textile surface of the insole.

If the insoles are too big, trim the front of the insole, using scissors: there are marks provided for this purpose.

Care

It is recommended to wash the product every day, or after each use. Hand wash. Do not dry clean. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not tumble-dry. Do not iron. Dry away from any direct heat source (radiator, sun, etc.). Dry flat.

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

This medical device is a regulated health product with CE marking

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©Thuasne - 2030802 (2022-07)
Thuasne SAS – SIREN/RCS Nanterre 542 091 186