THUASNE

Silistab® Malleo

fr	Chevillère élastique proprioceptive avec protection des malléoles.....	5
en	Proprioceptive elastic ankle brace with malleoli protection	6
de	Elastische propriozeptive Knöchelbandage mit Knöchelschutz	7
nl	Proprioceptieve elastische enkelbrace met enkelbescherming.....	8
it	Cavigliera elastica proprioceptiva con protezione dei malleoli.....	10
es	Tobillera elástica propioceptiva con protección de los maléolos.....	11
pt	Tornozoleira elástica propioceptiva com proteção dos maléolos	12
da	Proprioceptisk, elastisk ankelbind med beskyttelse af ankelknuderne.....	13
fi	Proprioseptiivinen joustava nilkkatuki, jossa kehräsluiden suoja.....	14
sv	Proprioceptiv elastisk ankelortos med skydd för fotknölarne	16
el	Ελαστική ιδιοδεκτική επιστραγαλίδα με προστασία των σφυρών.....	17
cs	Elastická proprioceptivní kotníková ortéza s ochranou kotníků	18
pl	Elastyczna proprioceptywna orteza skokowo-stopowa z zabezpieczeniem kostek.....	19
lv	Proprioceptīvs elastīgs potītes pārsējs ar potīšu kaulu aizsardzību.....	20
lt	Tamprus propriocepriją gerinantis čiurnos įtvaras su čiurnakaulių apsaugos zonomis.....	21
et	Proprioseptiivne elastne pahluuside malleoolikaitsmetega.....	23
sl	Elastična proprioceptivna opornica z zaščito za gleženj.....	24
sk	Elastická proprioceptívna členková ortéza s ochranou členkov	25
hu	Proprioceptív rugalmas bokarögzítő bokavédővel	26
bg	Проприоцептивна еластична шина за глезен със защита на малеолите.....	27
ro	Glezneră elastică proprioceptivă cu protecție pentru maleole.....	29
ru	Эластичный проприоцептивный ортез с защитой лодыжки.....	30
hr	Elastični proprioceptivni steznik za gležanj s maleolarnom zaštitom.....	31
zh	帶有踝骨保护的弹力本体感护踝.....	32
ar	دعامة كاحل مرنة متعلقة باستقبال الحس العميق مع حماية الأعقاب	35

	1	19 - 20 cm
	2	21 - 22 cm
	3	23 - 24 cm
	4	25 - 27 cm
	5	28 - 30 cm
	6	31 - 33 cm

fr	Tour de cheville
en	Ankle circumference
de	Knöchelumfang
nl	Omvang van de enkel
it	Giro caviglia
es	Circunferencia del tobillo
pt	Perímetro do tornozelo
da	Anklens omkreds
fi	Nilkanympärys
sv	Runt fotleden
el	Περίμετρος αστραγάλου
cs	Obvod kotníku
pl	Obwód kostki

lv	Potītes apkārtmērs
lt	Kulkšnies apimtis
et	Pahkluu ümbermõõt
sl	Obseg gležnja
sk	Obvod členka
hu	Boka kerülete
bg	Обиколка на глезена
ro	Circumferința gleznei
ru	Окружность голени
hr	Opseg gležnja
zh	腳踝尺寸

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en

PROPRIOCEPTIVE ELASTIC ANKLE BRACE WITH MALLEOLI PROTECTION

Description/Destination

This device is a proprioceptive elastic ankle brace providing joint support. The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Composition

Elastodiène - Viscose - Polyester - Elastane - Polyamide - Silicone - Cotton - Polyuréthane.

Properties/Mode of action

Elastic compressive knitting (class 3 – 20–36 mmHg / 27,1-48 hPa).
Silicone padding : Protection of the malleoli, and instep.

Indications

Mild ankle sprain (acute phase).
Post-operative, post-traumatic residual pain and/or edema.
Joint instability/laxity.
Proprioceptive support for rehabilitation.
Sport activities resumption.
Compression for scar/burns healing.

Contraindications

Do not use the product if the diagnosis has not been confirmed.
Do not apply the product in direct contact with broken skin or an open wound without an adequate dressing.
Do not use in the event of known allergy to any of the components.
History of venous or lymphatic disorders.

Precautions

Verify the product's integrity before every use.
Do not use the device if it is damaged.
Choose the appropriate size to fit the patient, referring to the size chart.
It is recommended that a healthcare professional supervises the first application.
Strictly comply with your healthcare professional's prescription and recommendations for use.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

For hygiene and performance reasons, do not re-use the product for another patient.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

If the support is worn for prolonged periods of time, ensure that blood circulation is not impeded.

Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.
Possible risk of venous thrombosis.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

Pull the ankle brace over the ankle joint, like a sock.
Place the malleoli correctly above the silicone insert.
The silicone insert should not lie on the malleoli.

Care

Product can be washed in accordance with the conditions shown on this notice and on the label. Machine washable at 40°C (delicate programme). Do not use detergents, fabric softeners or aggressive products (products containing chlorine, etc.). Do not dry clean. Do not tumble-dry. Do not iron. Squeeze out excess water. Dry flat. Dry away from any direct heat source (radiator, sun, etc.).

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.
Keep this instruction leaflet

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©Thuasne - 2023701 (2021-02)
Thuasne SAS - SIREN/RCS Nanterre 542 091 186 - capital 1 950 000 euros